

► **Characteristics**

- Patient-pleasing flavors.
- Flow-set formulation eliminates gagging.
- Formulation is smooth and pleasant consistency.

► **Components**

- Sodium fluoride : 27mg/g
- Phosphoric acid : 12mg/g

► **Indications**

- Tooth remineralization agent


► **Directions**

- Clean teeth and remove excess fluid.
- Fill applicator tray about 1/3 full with gel.
- Insert tray in mouth and have patient bite down lightly for 60 seconds or up to 4 minutes.  
(A slight chewing motion enhances interproximal coverage)
- Remove tray from mouth and have patient expectorate excess gel.
- Instruct patient not to eat, drink or rinse for 30 minutes.

► **Caution**

- Keep out of reach of children.  
(Special care should be used when administration to small children so that gel is not ingested.)
- For professional use only.  
(This product is not intended for home or unsupervised consumer use.)
- Avoid contacting with a plenty of water.
- Do not eat. If a large quantity is swallowed, induce vomiting by medical personnel.

Xylitol

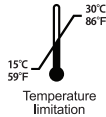

Manufactured by

 **VERICOM CO., LTD.**

48, Toegyegongdan 1-Gil, Chuncheon-Si,  
Gangwon-Do, Korea

<http://www.vericom.co.kr>

Made in Korea